

# Clip-Fix

## One-Piece Tube Holder

**Clip-Fix** is a Velcro® strap with a metal holder, which serves for the sorting of tube and cable systems at the bed sheet, patient's clothing or DIN rails.



**Clip-Fix** is available in three different versions and in different lengths:

<b>Clip-Fix 1</b>	REF 30 - 30 - 01	total length 11 cm	width 2,5 cm
<b>Clip-Fix 2</b>	REF 30 - 30 - 02	total length 17 cm	width 2,5 cm
<b>Clip-Fix 3</b>	REF 30 - 30 - 03	total length 23,5 cm	width 2,5 cm
<b>Clip-Fix 1 Duo</b>	REF 30 - 30 - 04	total length 12,5 cm	width 2,5 cm
<b>Clip-Fix 2 DIN</b>	REF 30 - 30 - 02D	total length 18,5 cm	width 2,5 cm
<b>Clip-Fix 3 DIN</b>	REF 30 - 30 - 03D	total length 25 cm	width 2,5 cm

### INSTRUCTION FOR USE - APPLICATION

Open the strap, put the tube to be fixed inside and press the strap end over the tube. Clamp Clip-Fix now by means of the metal holder at the bed sheet, the patient's clothing or DIN rails to its required position.

In the emergency rescue service **Clip-Fix** is an optimal facility to sort cables, e.g. EGG cable, to fix catheter or infusion bags in the mobile ICU vehicle as well as at the scene or when transporting patients.


In neonatology **Clip-Fix** allows mobility and offers a special safety to the parents when practicing kangaroo care with the little patient.

**Clip-Fix Duo** has the advantage that tubes can be fixed on both sides and thereby preventing them from shifting in one direction. It furthermore serves as a fixation for secretion bags at the patient's clothing.

The stainless steel clip of **Clip-Fix DIN** is standardised for all commonly used DIN rails with cross-sectional dimensions of 25 mm x 10 mm such as the Dräger rail.

### INFORMATION:

Reusable product, reuse is possible by hygienic reprocessing. All versions of **Clip-Fix** are washable up to 60 °C and can be disinfected by standard disinfectant or vapour sterilisation. The metal holders consist of an acid-resistant steel. In case of residual dirt and/or wear after hygienic reprocessing the fixation may no longer be used.

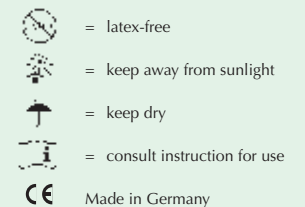

# Clip-Fix

## One-Piece Tube Holder - Examples of application



### Clip-Fix 1

Guiding and holding of several tube systems with fixation at the patient's clothing.



### Clip-Fix 1

Guiding and holding of several tube systems with fixation at the bed sheet.



### Clip-Fix 1

Guiding and holding of several tube systems with fixation at the patient's pillow.



### Clip-Fix 2

Fixation of the urinary bag at the stretcher in the rescue vehicle.



### Clip-Fix 3

Guiding and holding of the breathing tube and the cable systems at the service rail placed on the handgrip of the rescue vehicle.



### Clip-Fix 1 + Clip-Fix 2

Guiding, sorting and holding of the breathing tube and the cable systems at the service rail placed on the infusion stand.