

Eye-Care



NOVO

Novo Klinik-Service GmbH



Metra helps helping!



Place the eye protector over the child's eyes and loop the end without Velcro® around the head.



Close the eye protector by using the Velcro®.



Please use **Eye-Care Fix** additionally with agitated children.

Detach the Velcro® coin from one end of the head strip.



Put the detached Velcro® coin directly on the eye protector and fix the head strip then on the other side with the adherent Velcro® coin upon the eye protector.



Close the head strip. For maximum security we recommend to place the head strip transversely from the back third to the front of the head. If necessary, cut the excess part off.



Fixation of the eye protector with head strip in final position