

PulsOxy-Fix

non-adhesive



NOVO

Novo Klinik-Service Gml



Metra helps helping!

Fixation for Pulse Oximeter Sensors

PulsOxy-Fix is a softly padded fixation for pulse oximeter sensors, that is fixed on the skin non-adhesively. It can be applied at the foot, finger or other extremities.



PulsOxy-Fix is available in three sizes:

PulsOxy-Fix 10	REF 30 - 14	1,8 cm width	10 cm length
PulsOxy-Fix 12	REF 30 - 10	3 cm width	12 cm length
PulsOxy-Fix 17	REF 30 - 12	4 cm width	17 cm length

ADVANTAGES

safe, quick and comfortable handling – even with agitated children

eudermic – softly padded, no sharp edges

non-adhesive – soft to tender skin

latex-free – hypoallergenic, bacterial neutral, protection against decubitus ulcer

adjustable individually – can be cut to length on request

economic – can be opened and closed at any time

sterilisation is possible e.g. by vapour at 134 °C

Made in Germany

patient

nursing
staff

finance
controller



= do not reuse

= latex-free

= keep away from sunlight

= keep dry

= consult instruction for use

Made in Germany



PulsOxy-Fix

non-adhesive



NOVO

Novo Klinik-Service GmH



Metra helps helping!

Instruction for use



Open the sleeve, remove the cover of the adhesive layer and fix the pulse oximeter sensor on it.



Place the pulse oximeter sensor at the foot respectively at the desired extremity.



Wind the sleeve once around the sensor and close the sleeve with the Velcro®.



Fixation in final position.



PulsOxy-Fix 12
Fixation in final position.



PulsOxy-Fix 10
Fixation in final position.