

Soft-Fix

Non-adhesive Multi-Fixation

Soft-Fix is a non-adhesive, multifunctional fixation which is used for patients of all ages. It serves as fixation for infusion cannulae, probes, catheters and life keeping tube systems. Furthermore, it is used for splints, wound care and as decubitus prophylaxis.



Soft-Fix is available on rolls in three different widths and supplied in a handy dispenser box:

Soft-Fix small	REF 30 - 01	2 cm width, 5 m length
Soft-Fix normal	REF 30 - 02	4 cm width, 5 m length
Soft-Fix large	REF 30 - 03	6 cm width, 5 m length
Soft-Fix Velcro®	REF 30 - 04	2 cm width, 15 m length
Soft-Fix Mix	REF 30 - 05	5 x small, 2 x normal, 1 x large, 1 x Velcro®

The **white** foam side shows towards the patient or object so that the adherent effect, and therefore the optimal grip, is guaranteed!

Depending on the application, place **Soft-Fix** on the patient or at the object to be fixed and close the fixation with the Velcro® strip.
If necessary, cut into the fixation beforehand and build a loop.

ADVANTAGES

	patient	nursing staff	finance controller	
safe, quick and comfortable handling	●	●	●	
eudermic – non adhesive, soft and reclosable	●	●	●	
latex-free and non adhesive – hypoallergenic, bacteria-neutral, protection against decubitus ulcer	●	●	●	
low density of material (less than 30% bear on the skin)	●	●		☒ = do not reuse
reduction of tensile loading and tender spots	●	●		☒ = latex-free
slightly absorptive – no loss of function, readjustable at any time	●	●		☀ = keep away from sunlight
economic – can be cut according to the required length	●	●	●	☂ = keep dry
sterilisation is possible e.g. by vapour at 134°C	●	●	●	☒ = consult instruction for use
Made in Germany	●	●	●	CE Made in Germany



Soft-Fix


Instruction for Use



Cut the fixation band and the Velcro® off, according to the required length.



Cut a slot in the material close to one end of the fixation band.

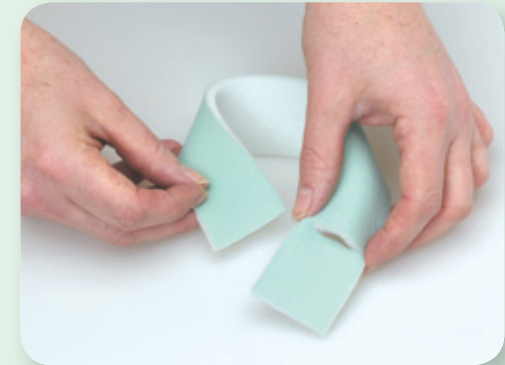

The band does not rip at the cut surface.



Put the other end through the slot to build a loop. The **white** foam side shows inwards so that the adherent effect, and therefore the optimal grip, is guaranteed!



Put the tube into the loop and tighten it. Tightening ensures a safe fixation.



Place the fixation band on the patient and fix it with the Velcro®.

Soft-Fix

Non-adhesive Multi-Fixation - Examples of application



Securing of infusion cannulae, catheters and arterial sphygmomanometers



Securing and padding of charges and discharges




Securing of ventricular drainages



Securing of venous drainages and infusion cannulae applied to parchment-like skin.



Securing of urinary catheters or in dialysis

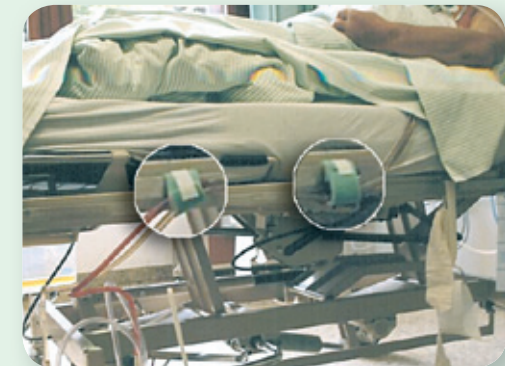

Securing of charging and discharging tubes e.g. at the bedside

Soft-Fix

Non-adhesive Multi-Fixation - Examples of application



Securing of charging and discharging tube systems as well as saturation tubes for premature infants and babies



Securing and cushioning of tubes and cannulae, e.g. femoral catheters or arterial accesses by using a **Soft-Fix** loop



In support of respiration with CPAP-masks – closure of the mouth with **Soft-Fix** small



Fast and skin-friendly fixation of saturation tubes



Securing and cushioning of tube systems



Fixation of pulse oxymeters by ambulatory care nursing of children