

TuBo-Clip non-adhesive

One-Piece Endotracheal Tube Holder

TuBo-Clip is a one-piece endotracheal tube holder for adults, adolescent patients and pupils as well as a fixation to stabilise tensile-loaded probes. It is used for oral and nasal application. Suitable for patients with skin irritations.



Outstandingly qualified for emergency medical services!

TuBo-Clip is available in two different sizes:

| | | |
|-----------------------------|-------------|--------------------|
| TuBo-Clip normal | REF 20 - 01 | total length 72 cm |
| TuBo-Clip extra long | REF 20 - 05 | total length 85 cm |

To guarantee the adherent effect and therefore the optimal grip, it is necessary to direct the **white** foam towards the patient's skin or the tube! The Velcro® adheres onto the **green** side.

ADVANTAGES

| | patient | nursing staff | finance controller |
|--|---------|---------------|--------------------|
| safe, quick and comfortable handling | ● | ● | ● |
| non-adhesive – ideal for patients with beards, skin burns or shock patients | ● | ● | ● |
| latex-free – hypoallergenic, bacterial neutral, protection against decubitus ulcer | ● | ● | ● |
| low density of material (less than 30% bear on the skin) | ● | ● | |
| slightly absorptive – no loss of function, readjustable at any time | ● | ● | |
| adjustable individually – fixation fits all patients | ● | ● | ● |
| applicable also for double lumen tubes and larynx-masks | | ● | ● |
| sterilisation is possible | ● | ● | ● |
| economic – optimal price-performance ration | ● | ● | ● |
| Made in Germany | ● | ● | ● |

TuBo-Clip

Application with Disconnected Tube

NOVO

Novo Klinik-Service GmbH

metra

Metra helps helping!



Slip the fixation band over the tube by using the slot.



Place the fixation band around the patient's neck with the **white** material facing the skin.



Wind the fixation band with a slight tug twice around the tube.



Tighten the end and fasten it with the Velcro® strip on the **green** fleece side. If necessary, cut the excess part off first. Fixation in final position



A Guedel airway can be fixed easily by wrapping it.



Fixation with Guedel airway in final position